


Quick Guide

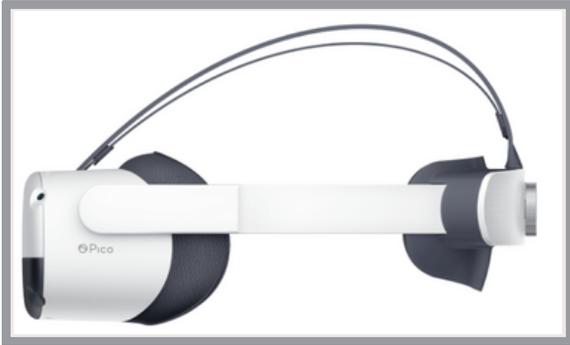

Headset (side view)




Headset (front view)



Hand Controller

Powering Headset On and Off

- Press and hold the Power Button to turn the headset on and off
- Easy to access USB-C charging port

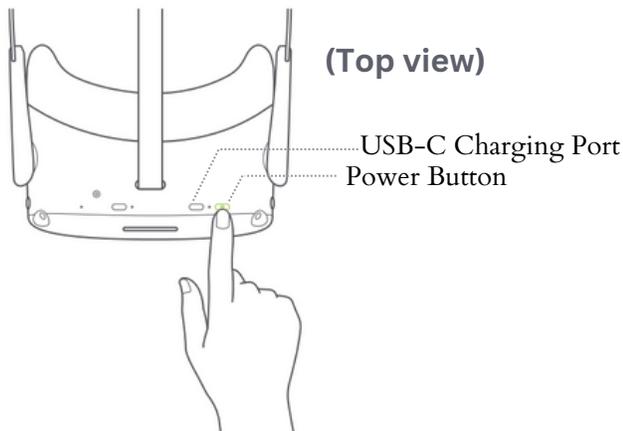

Hand Controller (Power)

- 1.5V AA Battery Powered
- Easy to change and install batteries

Pull the tab to remove the insulating paper by following the picture.



Slide the battery cover according to arrow direction with your palm.

Hand Controller (Main Buttons)



- Power Button




- Main Trigger is use during Visual Field testing (Using the index finger)
- All other buttons are **DISABLED** outside of the Main trigger and Power button.

Quick Guide



Easy to Clean Insert

- Easy to clean face insert
- Patient can wear their glasses or contact lenses during testing.

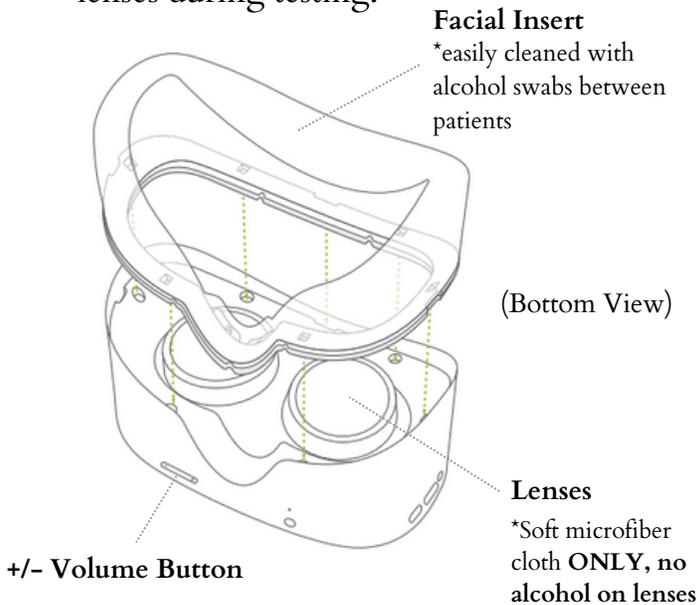


- The lenses are easily damaged if alcohol or other harsh chemicals are used on the lenses during cleaning.
- A soft microfiber cleaning cloth is recommended for cleaning the lenses. **No alcohol on the lenses.**
- Keep the lenses directly out of sunlight for prolonged periods.

***The use of alcohol or harsh chemicals will instantly damage the anti-reflective coating on the lenses, voiding the manufacturer warranty. Use of a soft microfiber cloth is recommended for lens cleaning.**


Comfortable and Adjustable fit

Turn the strap dial counterclockwise to loosen the VR Headset.
Rotate the strap up to ensure enough space for your head.

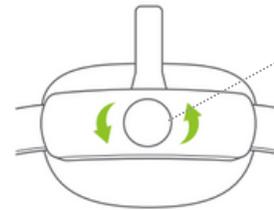

Turn the knob on the back of the headrest counterclockwise to make the strap larger to fit over the head.


Starting position

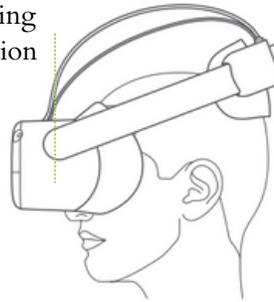


Start with the headrest in the up position and move it downward over the back of the head.

*Note: You can use this product with prescription glasses.

Securing the Headset

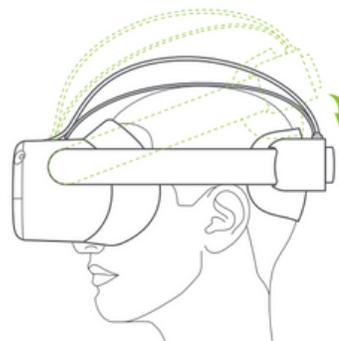

Rotate the strap down over head and turn the headrest knob clockwise to secure fit.


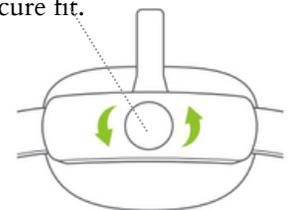


For service and Support contact us at support@seeheru.com



1-844-SEE-HERU (733-4378)

Creating New Patients, Appointments, and Reports in Heru Portal

Easily Create New Patients

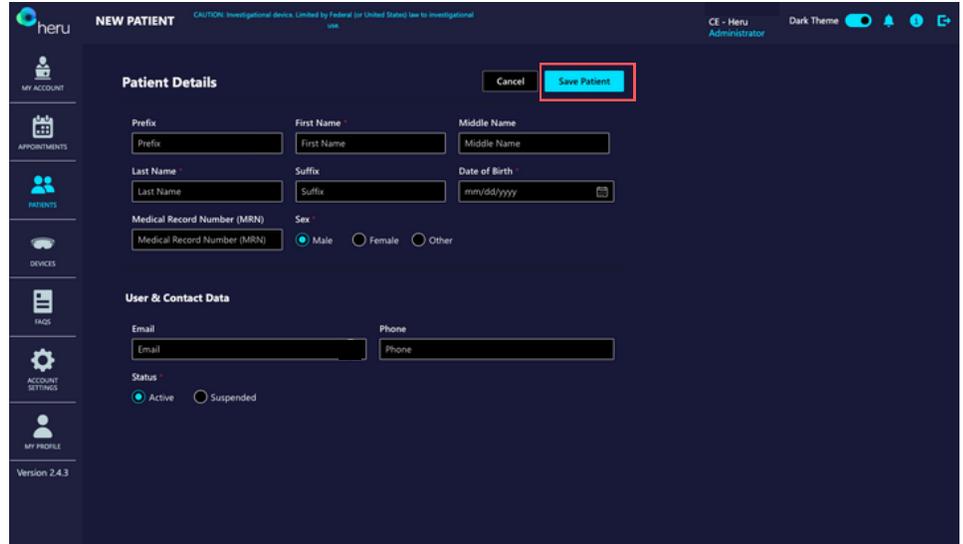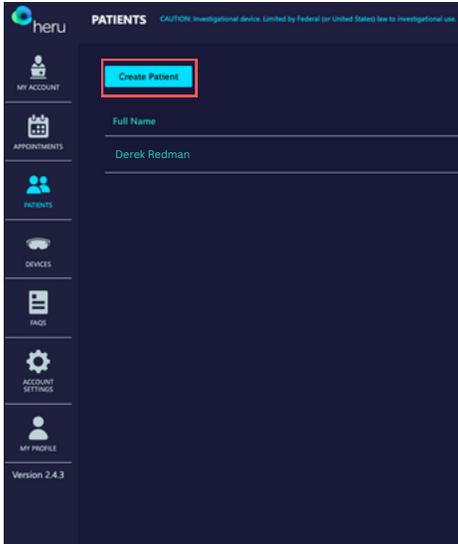


Set up Appointments in Advance

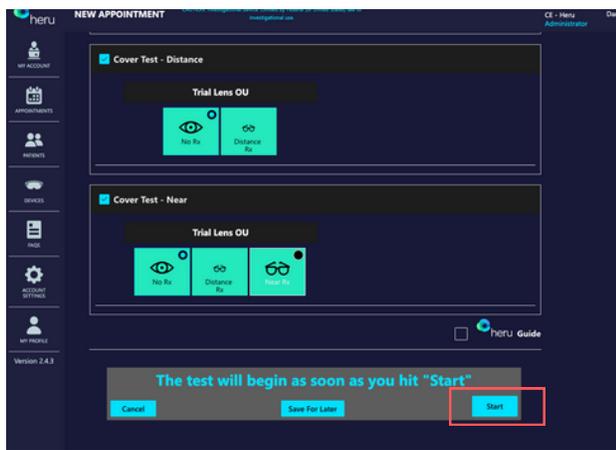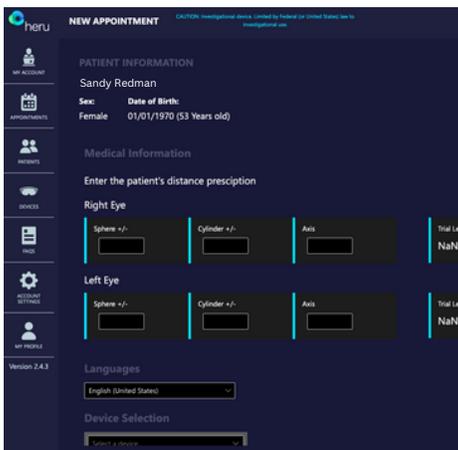


Patient will see the "H" before the test begins

Monitor Tests in Portal

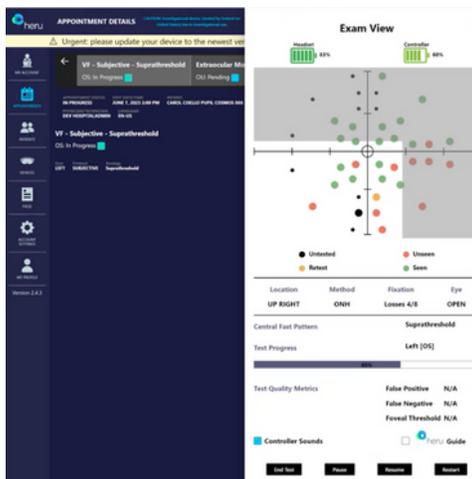


Easy Access Reporting

VF Report

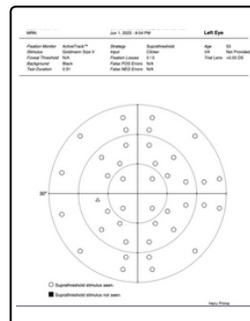

Pupils

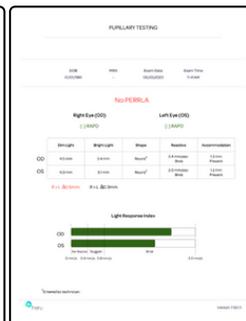

EOM



Cover



For service and Support contact us at support@seeheru.com
1-844-SEE-HERU (733-4378)